HUMANTECHNIK

time *flash*

DE

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Seite 2

Blitzwecker

EN

User manual

Page 8

Flashing alarm clock

FR

Mode d'emploi

Page 14

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NL

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Pagina 20

Flitswekker

IT

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Pagina 26

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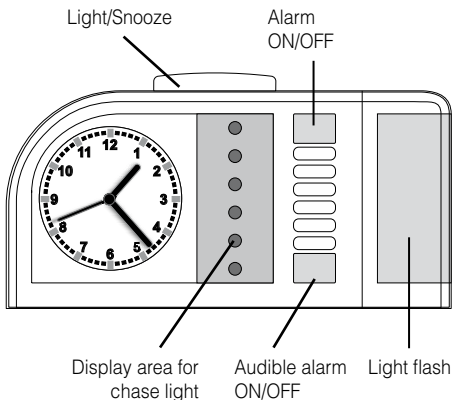
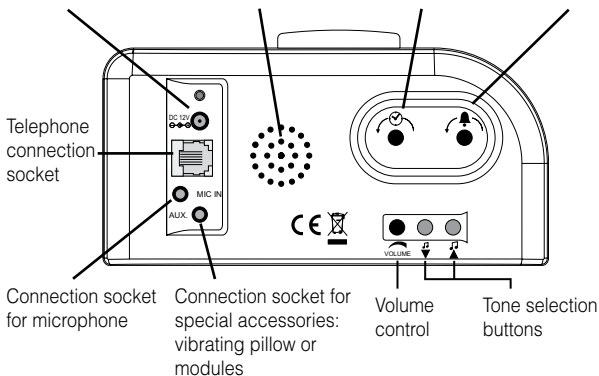
12 V-connection socket Sound outlet of the audible alarm Button for setting clock time Button for setting alarm time

*) Underneath the device: Dial switch for setting the microphone sensitivity

We would like to congratulate you on the purchase of our »time flash« alarm clock. You have chosen a modern and reliable system. Please read the operating instruction carefully to be able to start the unit correctly and to familiarize yourself with all of the system's features.

Standard components

Please check if all following components are included:

- »time flash«
- 12V power supply unit
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Functioning principle

The »time flash« is an electronic clock with an analogue time display. It wakes you up with a flashing light and/or an acoustic alarm.

Additionally it shows telephone rings by flash, blinking and sound signals.

The following special accessories can be connected to the alarm clock:

- Vibrating pad A-3303-0
- Vibrating contraption A-3310-0
- Telephone cable (optionally available)
- Microphone cable (3.5mm TRS-plug, optionally available)

Startup

Plug in the power supply unit of the »time flash« into a 230 V power outlet and connect the cable with the 12 V connection socket.

Time setting

Turn the button for setting the time  in the direction of the arrow until the desired time is set.

Alarm setting

Turn the button for setting the alarm  in the direction of the arrow until the required alarm time is set.

Switching on the alarm

Switch on the alarm using the ON/OFF switch.

Switch not pressed: Alarm switched on (green outline visible around switch).

Switch pressed: alarm switched off.

Switching on the audible alarm

Switch on the audible alarm using the audible alarm ON/OFF switch.

Switch not pressed: Audible alarm switched on (green outline visible around switch).

Switch pressed: Audible alarm switched off.

Select alarm tone/ melody

You can select the alarm tone/ melody with the tone selection buttons. You can choose from 10 different types of tone sequences.

Snooze Alarm

Press the Light/Snooze button to suspend an alarm which is in progress. The alarm is triggered again after about 5 minutes.

Telephone ring signalling

The »time flash« can show telephone rings by flash, blinking and tone signals.

There are two connection possibilities to pick up the telephone ring:

1. You can pick up the telephone ring signal by using a telephone cable (optionally available). This way you connect a special N-coded telephone cable with the Telephone socket.
2. You can pick up the ring signal from the sound outlet of the telephone by using an external microphone (optionally available) which is fixed with a self-adhesive pad to the sound outlet.

The microphone cable is connected to the "MIC IN" connection socket at the backside of the »time flash«.

Note:

The ringing signal of the telephone is set factory-wise and cannot be changed with the tone selection buttons.

Setting the sensitivity

Set the ringer of your telephone to its maximum volume. Set the sensitivity of the »time flash« to its lowest level (turn to the anticlockwise stop).

Activate the telephone ringer and gradually increase the sensitivity of the »time flash« (by turning the dial clockwise) until the »time flash« reacts to the telephone ringing.

Please note: It is possible that the telephone signal may be triggered once if you replace the telephone handset with too much force.

Switching on the clock face illumination

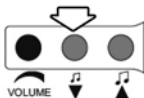
Press the Light/Snooze button to switch on the clock face illumination for about 6 seconds.

Switching the flash on and off

The flash feature of the timeflash alarm clock can be switched on and off.

To switch the flash off, please proceed as follows:


- Switch off the alarm and the tone (Switches are pressed in, the green edge is not visible).
- Now press the alarm repeat button on the top side of the alarm clock and keep it held down.
- Press six times on the left tone adjuster button (situated at the right side of the volume control).

When pressing the button you will hear a beep tone.

- Let go of the alarm repeat button. The flash is now switched off.

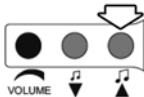To switch the flash on, please proceed as described above, but press six times on the right tone adjuster button.

The flash is now switched on again.

Special aspects

- The alarm signal is repeated after 5 minutes (autosnooze) unless you press the Light/Snooze button or the alarm ON/OFF button.
- The quartz mechanism of the alarm clock has a backup power supply. The clock will continue to operate for about 2 hours if there is a power failure. The alarm function and the illumination are deactivated during this period.
- The clock face illumination is also activated when the alarm is triggered.
- In addition to the green marking of the Alarm ON/OFF switch, the snooze button is also illuminated when the alarm is triggered.
- The »time flash« uses a chase light for signalling an alarm. During an alarm the 6 LEDs at the frontside will be switched alternately, creating the chase light.

Maintenance and care

The »time flash« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The »time flash« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electrical and electronic units (applicable in the countries of the  European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

Technical data

Power supply:	DC 12V / 1000mA
Power consumption:	approx. 2 W
Height:	86 mm
Width:	175 mm
Depth:	53 mm
Weight:	337 g

This device satisfies the following EC directives:

- 2011 / 65 / EC + 2015 / 863 / EC RoHS directive
- 2012 / 19 / EC WEEE directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com/service.

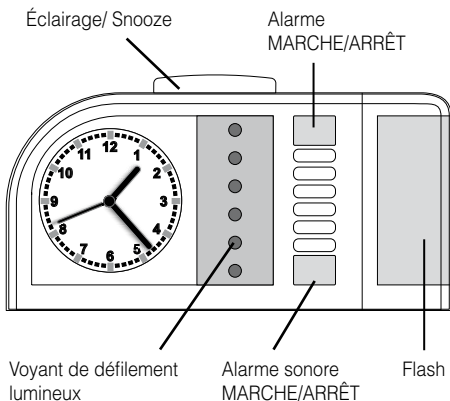
Humantechnik's UK authorised representative:

Sarabec Ltd.
15 High Force Road
MIDDLESBROUGH TS2 1RH
United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments. UK declaration of conformity available from: Sarabec Ltd.

Technical specifications subject to change without prior notice.

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