# earis®

## MAX

**DE**

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Seite 2

**EN**

**User manual**

Page 9

**FR**

**Mode d'emploi**

Page 16

**NL**

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Pagina 22

**ES**

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Página 29

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## Introduction

Our earisMAX Bluetooth headphones, equipped with Auracast technology, active noise cNCellation, and an advNCed hearing loss calibration system, offer an exceptional listening experience that redefines the boundaries of sound. With support for LDAC, the high-resolution audio codec, you'll experience sound quality that comes remarkably close to the original, immersing yourself in a world of clarity, detail, and personalized sound adjustment while enjoying our innovations in a comfortable and stylish design. Welcome to a new era of wireless music enjoyment, welcome to our earisMAX Bluetooth headphones with Auracast and hearing loss calibration system.

Please read this user manual carefully to properly operate the device and become familiar with all the features it offers.

## Package Contents

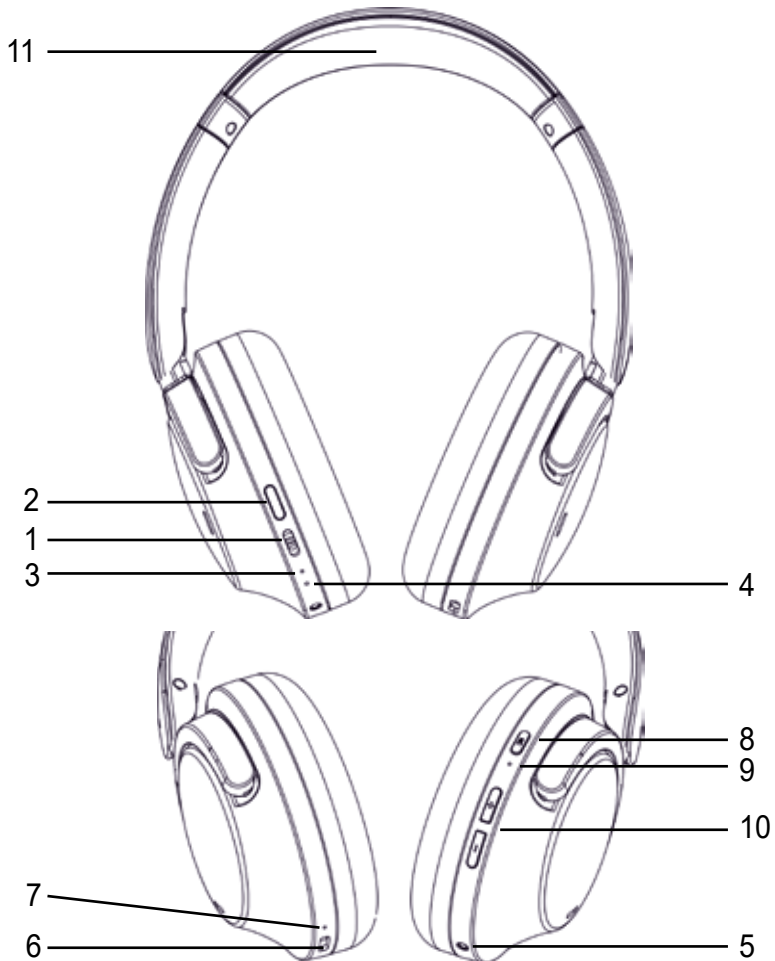
- earisMAX headphones
- USB-A to USB-C charging cable
- Hard case
- Airplane Headphone Adapter
- 3.5mm connection cable
- Quick start guide
- Safety instructions
- Warranty Card

## earisMAX App

With the earisMAX app, you can individually configure your earisMAX headphones. Download the earisMAX app to your smartphone. The app is available for both Apple iOS and Android and can be downloaded from the respective app stores.

## Product Overview

1. Power On/Off Switch

2. Function Button

3. Status LED

4. Microphone

5. 2.5mm Audio socket for  
Audio Cable

6. USB-C Port for Charging Cable

7. Charging Status LED

8. NC Control Button

9. NC Control LED

10. Volume Buttons

11. Zone for Wireless Charging

## Turning On and Off the Headphones

You can turn the headphones on and off using the Power On/Off Switch (1). The Status LED (3) will start blinking white, and the announcement „Power on“ will be made. Turning off the headphones will be confirmed by the announcement „Power Off.“

## Pairing with a Playback Device

Before using your earisMAX for the first time, we recommend pairing it with your mobile phone or another device via Bluetooth®. The exact procedure for pairing may vary depending on your playback device. We recommend consulting the user manual of your playback device for the correct pairing steps.

### Bluetooth® Classic

If no connection is found after turning on the earisMAX, it will automatically switch to pairing mode after a few seconds. The Status LED (3) will blink alternately white and orange (announcement: Pairing).

If you want to connect earisMAX to another device, press the Function Button (2) for 5 seconds until the announcement „disconnected“ is made, followed by „pairing.“ The Status LED (3) will blink alternately white and orange. Earis MAX is now ready to pair.

Turn on the Bluetooth® function on your playback device and search for available devices if necessary.

In the Bluetooth® settings of your playback device, please select „earisMAX“ from the list of available devices and connect both devices. The Status LED (3) will blink regularly white (announcement: „Connected“). Automatic connection of both devices will occur during future power-ups.

### Auracast

If you turn on the earisMAX and no connection is found, it will automatically switch to pairing mode after a few seconds. The Status LED (3) will blink alternately white and orange (announcement: „Pairing“). If an Auracast channel is nearby, earisMAX will automatically connect to that channel. The Status LED (3) will blink regularly white. Once earisMAX is connected to your smartphone, you can select and play the desired Auracast channel using the earisMAX app. The Status LED (3) will blink regularly white.

## **Multipoint Connection**

EarisMAX can be connected to 2 Bluetooth® Classic devices. However, playback will occur from only one device at a time. To pair earisMAX with a second device, follow the instructions „Pairing with a Playback Device“ in this user manual.

EarisMAX stores the two most recent connections, allowing you to switch between them without reconnecting. You can disable multipoint connection in the earisMAX app.

## **Media Playback**

Please note that the functions described below may depend on the playback device and player software you are using and may not be available. EarisMAX only sends control signals compatible with Bluetooth®.

- **Play / Pause**  
Press the Function Button (2).
- **Previous Track**  
Press and hold the Volume Button (10) „up“ for 2 seconds.
- **Next Track**  
Press and hold the Volume Button (10) „down“ for 2 seconds.
- **Volume**  
Use the Volume Button (10) to adjust the desired volume.
- **Call Control**  
To answer a call with earisMAX, press the Function Button (2). To end the call, press the Function Button (2) again.

## **Active Noise CNCellation (NC)**

Enjoy your audio entertainment undisturbed in a noisy environment. You can activate Active Noise CNCellation (NC) by pressing the NC Control Button (8) briefly (announcement: „Noise cNCeling“).

## **Transparency Mode**

Experience your preferred audio entertainment while remaining attentive to important ambient sounds. You can activate Transparency Mode by pressing the NC Control Button (8) again briefly (announcement: “Ambient sound”). In Transparency Mode, ambient sounds are mixed with the music, allowing you to hear conversations or announcements.

## Turning Off Active Noise Cancellation

You can turn off Active Noise Cancellation (NC) and Transparency Mode by pressing the NC Control Button (8) briefly (announcement: “Noise canceling off”). In the earisMAX app, you can reduce the strength of noise cancellation and also switch more quickly between your preferred modes.

## Wear Detection

Media playback is paused when you stop wearing earisMAX and resumes when earisMAX is put back on. You can disable or enable wear detection in the earisMAX app.

## TailorID 2.0

In the earisMAX app, you can calibrate your hearing with the TailorID 2.0 feature to further enhance your listening experience.

## LDAC®

LDAC® (Lossless Digital Audio Codec) is a high-quality wireless audio compression technology developed to provide premium sound quality over Bluetooth connections. With LDAC®, you can wirelessly transmit high-resolution audio files from your audio device to your headphones or speakers. LDAC® offers high transmission rates and minimizes quality loss to ensure that your music or audio content is reproduced in stunning clarity and low latency.

Please note that your audio devices must support LDAC® for you to utilize this technology. Make sure to activate the LDAC® option on your devices if you want to enjoy the best possible sound quality.

You can enable or disable LDAC® in the earisMAX app.

**WARNING:** When LDAC® is activated, earisMAX will restart.

## Using the Audio Cable

If the battery is empty or for any other reason, the earisMAX device can be operated using the provided audio cable. Simply plug the cable into the 2.5mm jack socket (5) and connect it to the desired audio device.

## Factory Reset

To reset earisMAX to its factory settings, press the NC Control Button (8) and the Function Button (2) for 5 seconds.

## Charging earisMAX

You can check the battery status in the earisMAX app.  
earisMAX offers two charging options:

### USB-C

Connect the USB-C port (6) to the included USB-C cable with a USB charger. During charging, the Charging Status LED (7) next to the port will light up red. When the LED is off, the headphones are fully charged.

### Wireless Charging

earisMAX can be wirelessly charged using the optional earis MAX Charging Station via induction. The charging status will be displayed on the charging station.

## Technical Specifications

Frequency Range:	20Hz - 20kHz
Max. Sound Pressure Level (at 1 kHz):	123dB
Acoustic THD (at 1kHz):	<0.5%
RF Transmission Frequency Range:	2,40 – 2,4835 GHz
Transmission Power in mW/dBm:	< 10 mW/< 10 dBm
Operating Time per Battery Charge:	approximately 80 hours
Rechargeable battery capacity:	720mAh
Charging Time:	approximately 2 hours via USB-C
Operating Temperature:	0°C to 40°C
Weight:	250g

These devices comply to the requirements of the following EC directives:

- 2011/ 65/EU + 2015/863/EU RoHS-directive
- 2012/19/EU WEEE-directive
- 2014/53/EU RED-directive

Conformity with the above-mentioned directives is confirmed by the CE mark on the device. CE declarations of conformity are available on the Internet at **[www.human-technik.com/service](http://www.human-technik.com/service)**

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Sarabec Ltd