HUMANTECHNIK

*lisa*

**DE**

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Seite 2

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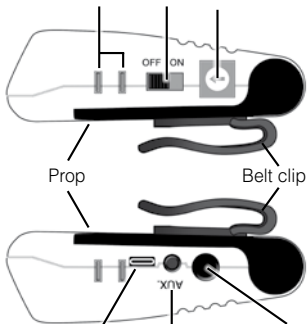
Charging contacts      On/Off      Channel selector

Prop

Belt clip

Charging socket      AUX-connector      Reset button

Indicator  
LEDs,  
event  
icons

Low  
battery  
indicator

Battery  
compartment

Congratulations on purchasing your »lisa-RF pager«!

Please read this manual carefully. It will tell you how to set up the system correctly and explain all the features it has to offer.

## **Contents of package**

Please check if the following parts are included:

- »lisa-RF-pager«
- neck cord
- 2 x 1.5 V AAA batteries
- User manual
- Warranty card

If any parts are missing, please contact your dealer or the manufacturer immediately.

## **Function**

The portable »lisa-RF-pager« is a receiver for the alerts of the various »lisa-RF« transmitters and indicates these as vibrations.

6 indicator LEDs with symbols help to distinguish the alert source.

## **Controls and displays**

Event indicator LEDs: The 6 indicator lamps help you to differentiate between the individual transmitter signals. They light up for a total of 40 seconds.

Low battery indicator: This LED lights up in blue to indicate that the battery is almost depleted. Please replace the batteries as soon as possible.

Reset button: When you feel the portable RF receiver vibrating, you can switch off its vibration using the reset button on the receiver. The event indicators light up and go out automatically after a total of 40 seconds.

Furthermore, you can press the reset button to recall the most recent signal again (the function works for up to 3 minutes after receiving the signal).

Belt clip: You can use the belt clip to attach the portable RF receiver to your clothing.

On/Off-Switch: This switch turns the portable RF receiver on and off.

Channel selector: You can select the transmission radio channel using the channel selector.

### **USB-C charging socket:**

There is a USB-C charging socket on the side

of the receiver. If the receiver is equipped with rechargeable batteries (accessory: 2 x A-4966-0, also included in the A-2910-0 charging station set), the receiver can also be charged using a standard USB charger with a USB-C cable instead of the charging station.

During charging, the battery status indicator lights up red. When charging is complete, the indicator goes out.

The maintenance function of the charge control automatically keeps the batteries at full capacity. This means that the charging process can be active again at irregular intervals, with the battery status indicator lighting up red accordingly.

When operating with normal batteries this charging socket remains without function.

## **Setting up the device**

### 1. Insert the batteries

The battery compartment is located on the underside of the portable RF receiver. Open it by sliding the cover of the battery compartment away from the receiver using both your thumbs. Insert two 1.5 V AAA batteries and refit the cover again.

## 2. Setting the correct radio channel

A digital transmission process ensures that the radio signals are received without errors.

This process also makes it possible to set 10 different radio channels. All RF lisa dices are set to channel 0 by default.

It is only necessary to set your system to another channel if you have several different users operating lisa units in proximity.

Set the required channel by using a small screwdriver to turn the channel selector switch, which is located on the right of the receiver, to the required channel number. Use a small screwdriver to adjust this switch.

All transmitters and receivers must be set to the same channel.

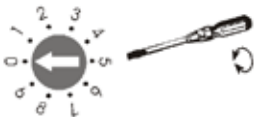
Fig.: setting the correct radio channel.

## **Operation**

### 1. Switching the receiver on

Slide the on/off- switch to »on«. The portable RF receiver is now ready to operate. After the

device has been switched on, each one of the event indicators lights up briefly in sequence and the receiver vibrates. This is a self-test of the device and lets you know that it is functioning correctly.

## 2. Event identification

The portable RF receiver displays the various incoming signals as coloured lights and different sequences of vibration.

Telephone: Telephone LED (orange) is lit.

Door 1: Door LED (green) is lit.

Door 2: Door LED (green) is flashing.

Baby monitor: Baby-cry-LED (white) is lit.

Call button: Person calling-LED (yellow) is lit.

Alarm: All LEDs are flashing.

Water alert: Water alert LED (blue) is lit.

Fire alert: Fire alert LED (red) is lit.

### Legend for vibration patterns:

V4 Vibration 0,4 seconds	P8 Pause 0,8 seconds
P12 Pause 1,2 seconds	P16 Pause 1,6 seconds
P20 Pause 2,0 seconds	P30 Pause 3,0 seconds
P36 Pause 3,6 seconds	P48 Pause 4,8 seconds

The event indicator LEDs remain lit for 40 seconds. The vibration stops after 28 seconds.

### 3. Recalling the most recent signal

The portable RF receiver stores each signal it receives for a period of 3 minutes.

During this time, you can recall the signal again by pressing the reset button. The event is then indicated by the indicator LEDs.

The vibration is not retriggered.

## Battery depleted

The blue battery indicator is lit. Please replace the batteries as soon as possible.

## Transmission range

Note the following points when using the portable RF receiver:

The radio signals are designed to have a range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Obstacles like building structures or vegetation.
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, these may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to a weakened or even cancelled-out signal by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.

- Other devices operating at similar frequencies in the vicinity may also result in mutual interference.

### **Special accessories**

The following modules (not included) can be plugged into to the AUX output connector:

- Vibrating pillow (A-3303-0)
- Flash module MF-1 (A-3320-0)
- Acoustic module MA-1 (A-3340-0)
- Vibrating contraption (A-3310-0)

Please note: The internal vibration is deactivated if the receiver is charged over the USB connector or the charging station. The AUX output remains active during charging.

## Maintenance and care

All »lisa-RF« devices are maintenance-free. If the units become dirty, simply wipe them clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Please do not expose »lisa-Funk« devices to direct sunlight for a long time and please protect them from excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

## Warranty

All lisa-Funk devices are very reliable products. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge.

It is recommended that you send in the product in its original packaging, so do not throw the

packaging away while the warranty period lasts. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal of used electric and electronic devices** (applicable in the countries of the European Union and other European countries with a separate collection system).

stem).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of these products.

Environment and health are endangered by a faulty disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

This device complies with the following EC directives:

- 2011 / 65 / + 2015 / 863 / EC RoHS directive
- 2012 / 19 / EC WEEE directive
- 2014 / 53 / EC RED directive

Compliance with the directives listed above is confirmed by the CE seal on the device.

CE compliance declarations are available on the Internet at [www.humantechnik.com/service](http://www.humantechnik.com/service).

Technical specifications subject to change without prior notice.

## Specifications

Power supply: 2 pcs 1.5 V AAA battery or  
1.2 V rechargeable battery

Battery endurance:

approx. 800 h with 1.5 V alkaline battery

approx. 400 h with 1.5 V zinc carbon battery

approx. 300 h with 1.2 V rechargeable battery

Frequency: 868,35 MHz

Vibration: Eccentric motor, various  
vibration sequences for identifying alerts

Height: 85 mm

Width: 60 mm

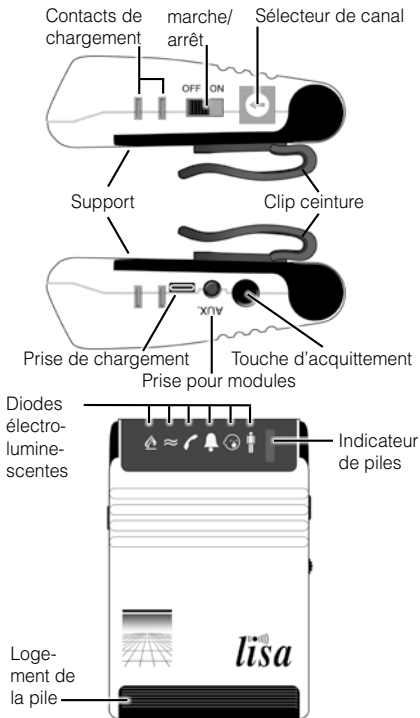
Depth: 28 mm

Weight: approx. 88 g

Operating range: 0° C – 40° C

Specifications subject to change without  
prior notice.

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**Humantechnik GHL AG**
**Sarabec Ltd.**