DE

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Seite 2

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EN

Instruction manual

Page 10

signolux receiver alarm clock

FR

Mode d'emploi

Page 18

Réveil *signolux*

NL

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Pagina 26

Digitale Flitswekker *signolux*

IT

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Pagina 34

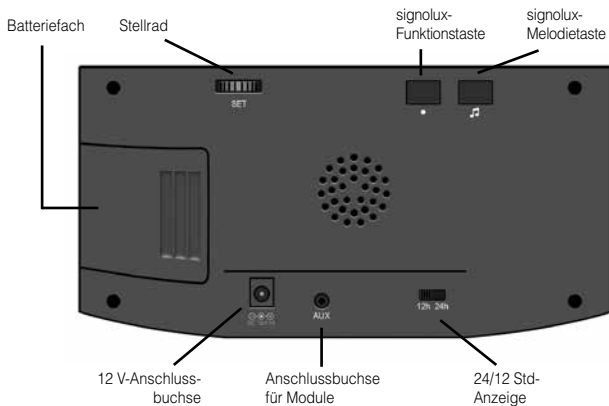
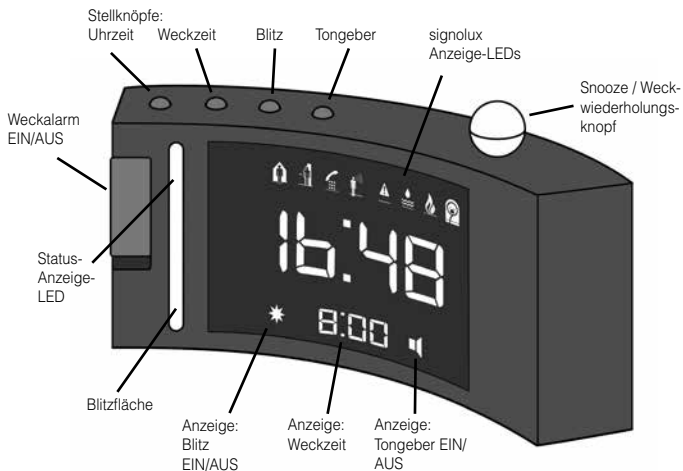
Sveglia digitale *signolux*

ES

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Página 42

Despertador receptor *signolux*

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Türklingel 1	LED <input type="checkbox"/>		20 Sek 10 Sek
LED	<input type="checkbox"/>		1 Sek 1 Sek
Vibration*	<input type="checkbox"/>		an Wiederholung aus
Türklingel 2	LED <input type="checkbox"/>		
LED	<input type="checkbox"/>		
Vibration*	<input type="checkbox"/>		an Wiederholung aus
Telefon	LED <input type="checkbox"/>		
LED	<input type="checkbox"/>		
Vibration*	<input type="checkbox"/>		an Wiederholung aus
Personenruf	LED <input type="checkbox"/>		
LED	<input type="checkbox"/>		
Vibration*	<input type="checkbox"/>		an Wiederholung aus
Alarm	LED <input type="checkbox"/>		
LED	<input type="checkbox"/>		
Vibration*	<input type="checkbox"/>		an Wiederholung aus
Wasseralarm	LED <input type="checkbox"/>		
LED	<input type="checkbox"/>		
Vibration*	<input type="checkbox"/>		an Wiederholung aus
Feueralarm	LED <input type="checkbox"/>		
LED	<input type="checkbox"/>		
Vibration*	<input type="checkbox"/>		an Wiederholung aus
Baby	LED <input type="checkbox"/>		
LED	<input type="checkbox"/>		
vibration*	<input type="checkbox"/>		on repeat off

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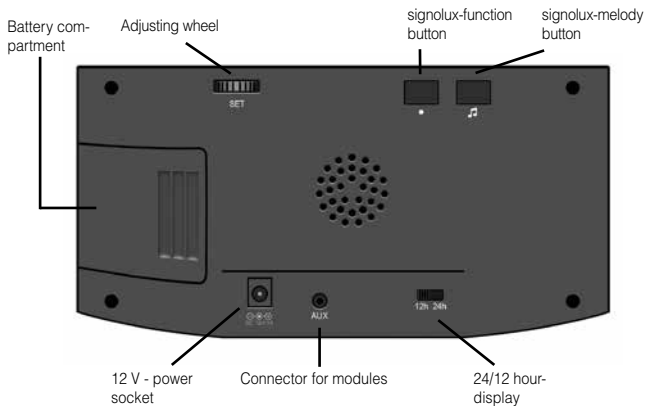
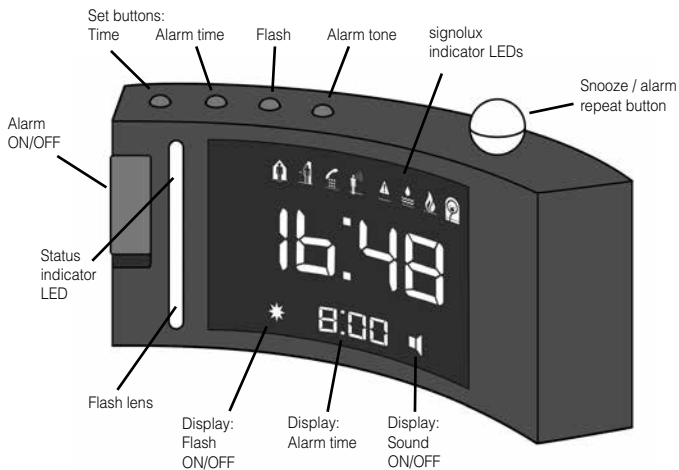
We would like to congratulate you on purchasing the »*signolux* receiver alarm clock«. You have chosen a modern and reliable system. Kindly read the instruction manual carefully in order to use the device properly and to familiarise yourself with all the possibilities of the system.

Package contents

Kindly check if all following parts are included:

- »*signolux* receiver alarm clock«
- 12 volts power supply unit
- Instruction manual
- Warranty card

Should any parts be missing, kindly contact your specialist dealer or the manufacturer directly.

Functioning principle

The »*signolux* receiver alarm clock« is an electronic clock with a digital time display. It wakes you up with an acoustic signal which is adjustable in pitch and volume. It is also a complete *signolux* receiver, capable of displaying the signals of all *signolux* transmitters. A received signal is displayed with flash and/or sound and identified by 8 indicator LEDs.

The following accessories can be connected to the »AUX« connector of the alarm clock:

- | | |
|--------------------|----------|
| - Vibrating pillow | A-3303-0 |
| - Flash module | A-3320-0 |
| - Acoustic module | A-3340-0 |

Setup

Connect the power supply unit to the 12 V power socket of the alarm clock and plug it into a 230 V power outlet.

Setting the time

To set the time, press the time set button »« for one second. The hour digits will start to flash. Set the desired hour by turning the adjusting wheel (on the reverse side of the of the clock) to the left or right.

Press the time set button »« again. The minute digits will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the time set button »« again will end the time setting procedure and the set time is displayed.

Setting the alarm time

To set the alarm time, press the alarm time button »« for one second. The hour digits of the alarm time will start to flash. Set the desired hour by turning the adjusting wheel to the left or right.

Press the alarm time button »« again. The minute digits of the alarm time will start to flash. Turn the adjusting wheel to the left or right to set the desired minutes.

Pressing the alarm time button »« again will end the alarm time setting procedure and the set alarm time is displayed.

Alarm ON / OFF

The waking alarm can be switched on and off with the slider switch on the left side of the alarm clock.

Slider in upper position (green area is visible): Alarm is switched ON.

Slider in lower position: Alarm is switched OFF.

Alarm tone ON / OFF

Press the alarm tone button »« for one second to switch the alarm tone ON or OFF. If the alarm tone is on, the speaker symbol will be visible in the display.

Setting the alarm tone volume

Press and hold the alarm tone button »« while adjusting the volume of the alarm tone. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button pressed and turn the adjusting wheel to the left or right to adjust the desired alarm tone volume. Release the alarm tone button to end the volume setting procedure.

Setting the alarm tone pitch

While adjusting the alarm tone pitch, press and hold both the alarm tone button »« and the spherical snooze button at the same time. After 2 seconds, the alarm tone can be heard. Keep the alarm tone button and the snooze button pressed and turn the adjusting wheel to the left or right to

adjust the desired alarm tone pitch. Release the buttons to end the pitch setting procedure.

Flash ON / OFF

Press and hold the flash button »« for one second to switch the flash ON or OFF. If the flash is on, the flash symbol will be visible in the display.

Setting the nighttime display brightness

The brightness of the displayed digits is automatically adjusted according to the light conditions within the room. During the day, the digits are displayed with full brightness. In darkness, the set nighttime display brightness is activated.

To set the nighttime display brightness, press and hold the flash button »«. Keep the flash button pressed and turn the adjusting wheel to the left or right to adjust the desired display brightness for the night. Release the button to end the setting procedure.

If this adjustment is done during daylight conditions, the display will switch back to full brightness after a few seconds.

Snooze function

You can interrupt a current alarm by pressing on the spherical snooze button. The alarm will be repeated after 5 minutes.

12 / 24 hour mode

The time can be displayed in a 12- or 24 hour mode.

The adjustment is made with the »24h/12h« switch on the reverse side of the alarm clock.

Power cut backup

The battery compartment can hold 4 batteries of size AAA (not included). In the event of a power cut, these guarantee the continuation of the set time and the display of the *signolux* signals.

Please note: During a power cut, the time display and the flash are not active as the power consumption is too high for the backup batteries!

Setting and programming

Programming signals and assignment to the symbols:

Up to 8 transmitters (e.g. the doorbell push-button) can be assigned to each symbol.

This assignment can be adapted by programming.

Kindly keep in mind that a transmitter can only be programmed if it is not yet known in the memory of the *signolux* receiver. If required, you first might have to delete the transmitter from the memory.

a) Keep the function button »●« pressed for some time. The symbols flash in sequence. Release the but-

ton as soon as you see the desired symbol.

b) Now trigger the signal using your transmitter (e.g. operate the doorbell pushbutton).

c) Wait for about 20 seconds and test the function.

Deleting transmitters from the receiver memory

Do the following if you want to remove transmitter from the memory:

a) Keep the function button »●« pressed for some time. The symbols flash in sequence. Let go of the button as soon as you see the desired symbol.

b) Now press the function button »●« and the melody button »♪« until all symbols briefly light up.

Please note that this will remove all assigned transmitters from the selected symbol.

Setting the melody

You can choose the melody / sound effect for each symbol:

- First trigger a signal (e.g. operate the doorbell pushbutton).
- The symbol assigned to the transmitter should light up and the previously chosen melody should be heard.
- Press and hold the melody setting button for 2 seconds. The symbol starts to blink.
- Repeatedly press the melody setting button »« until you have found the desired melody / sound effect (you can choose from 15 different sounds).
- Once the symbol ceases to flash, the selected melody has been assigned.

Meaning of the symbols

Each symbol can be assigned to a transmitter (e.g. the doorbell-push-button) and can be reprogrammed as preferred.

The symbols stand for:

	Doorbell 1	green light
	Doorbell 2	green light
	Telephone	orange light
	Person call	yellow light
	Baby	white light
	Fire alarm	red light
	Water alarm	blue light
	Alarm	red light

Transmitter battery status indicator:

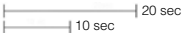
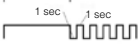
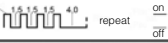
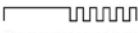
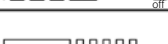
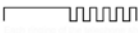
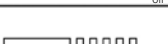
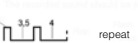
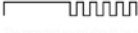
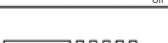
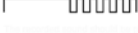
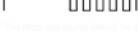
The receiver alarm clock informs about the state of the batteries within the *signolux* transmitters. The red status LED is located in the upper quarter of the flash lens.

A low battery level of a transmitter is indicated by the red control lamp shining together with the symbol of the according transmitter when it's signal is triggered.

Please replace the batteries of the according transmitter as soon as possible.

Signal distinction

* only with attached vibrating pad

Doorbell 1	 <p>20 sec 10 sec</p>
LED	 <p>1 sec 1 sec</p>
vibration*	 <p>1 1.5 1.5 1.5 4.0 repeat <input type="checkbox"/> on <input type="checkbox"/> off</p>
Doorbell 2	
LED	
vibration*	 <p>1 2.0 2.0 2.0 4.0 repeat <input type="checkbox"/> on <input type="checkbox"/> off</p>
Telephone	
LED	
vibration*	 <p>1 1.5 3.0 repeat <input type="checkbox"/> on <input type="checkbox"/> off</p>
Person call	
LED	
vibration*	 <p>1 1 1 repeat <input type="checkbox"/> on <input type="checkbox"/> off</p>
Alarm	
LED	
vibration*	 <p>1 3.5 4 repeat <input type="checkbox"/> on <input type="checkbox"/> off</p>
Water alarm	
LED	
vibration*	 <p>1 1 2.0 1 2.0 repeat <input type="checkbox"/> on <input type="checkbox"/> off</p>
Fire alarm	
LED	
vibration*	 <p>1 1 2.0 1 1 1 repeat <input type="checkbox"/> on <input type="checkbox"/> off</p>
Baby	
LED	
vibration*	 <p>1 2.0 2.0 1.5 1.5 1.5 repeat <input type="checkbox"/> on <input type="checkbox"/> off</p>

Transmission range

The signal range of the *signolux* transmitters can be as much as 200 metres under ideal conditions. The following conditions may result in a reduced range:

- Buildings or vegetation.
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter or receiver at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Maintenance and care

The »*signolux* receiver alarm clock« does not require any maintenance. If it becomes dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not expose it to full sunlight.

In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is not protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, above or near the product.

Warranty

The »*signolux* receiver alarm clock« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by unauthorised persons (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product serial number in any event.

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, our communal disposal company or your local dealer.

Technical data

Power supply:	DC12V /1000 mA
Power consumption:	approx. 2 W
Height:	103 mm
Width:	212 mm
Depth:	70 mm
Weight:	337 g

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments.

The full declarations of conformity for this device are available from Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH. UK.

Sarabec's EU authorised representative:

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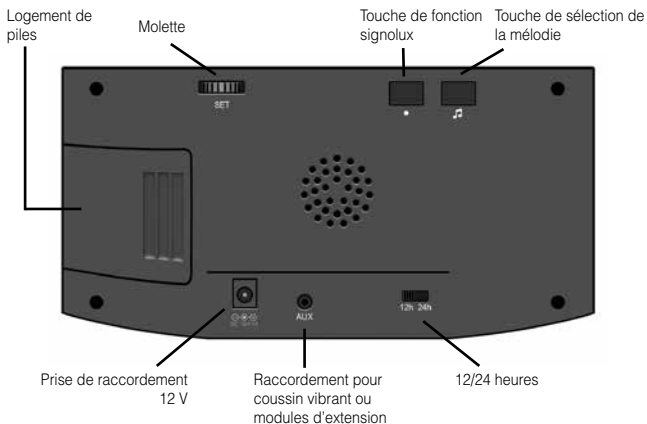
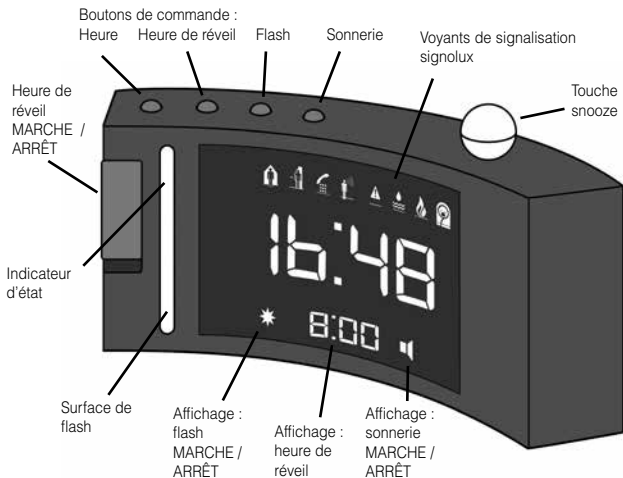
This device complies with the following EC directives:

- 2011 / 65 / EC RoHS directive
- 2012 / 19 / EC WEEE directive
- 2014 / 53 / EC RED directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com/service.

Technical specifications subject to change without prior notice.

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