HUMANTECHNIK

shake:up

DE

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Bluetooth®-Vibrationskissen

EN

Quick guide

Bluetooth® vibrating pad

FR

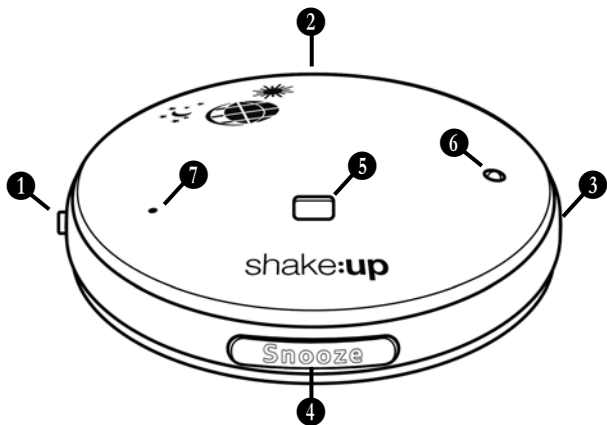
Guide rapide

Coussin vibrant Bluetooth®

NL

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Bluetooth®-trillingskussen

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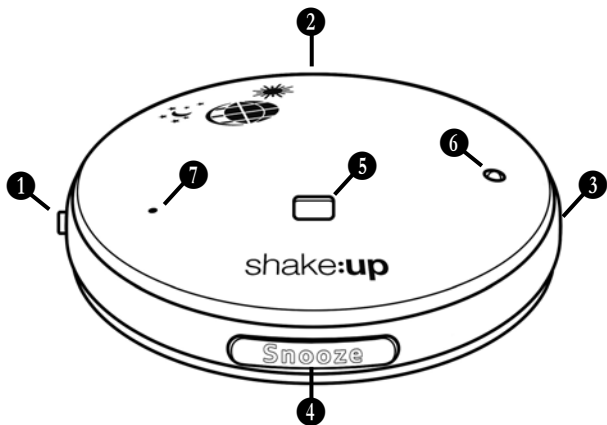
- 2 Lanyard fixation point
- 4 Snooze button
- 6 Status LED

Thank you for purchasing the programmable Bluetooth® vibrating pad »shake:Up«. Please read this manual carefully, so that you can familiarise yourself with all the possibilities this device offers.

Package contents

Kindly check if all following parts are included:

- »shake:Up« vibrating pad
- USB-C to USB-A charging cable
- Lanyard with safety connector (already attached)
- User manual
- Warranty card

Should any parts be missing, kindly contact your point of purchase or the manufacturer directly.

Setup

Please charge the device fully before use. Connect the pad with a USB charger (not included) or computer, using the included USB charging cable. Depending on the power output of the source, the device should be charged after approx. 3 hours. While charging, the status LED (6) lights up in red. When fully charged, the LED lights up in green, until you disconnect the charging cable. The LED will then turn off.

App installation and Bluetooth connection

Please locate the app »HT-Connect« in your respective app store (available for Android™ and iOS) and install it. Make sure to activate the Bluetooth function on your mobile phone or tablet. Open the »HT-Connect« app on your phone or tablet. Follow the instructions on the app.

Open the »HT-Connect« app on your phone or tablet. The pad should be listed on the menu - connect with it. The pad can now be programmed with various wake-up times and other options can be set. Each successful change is confirmed by the status LED lighting up in white. Under settings , please give the necessary permissions for further functions of the app.

Wake-up alarms

Use the controls on the app to set up various wake-up alarms according to your needs. The days of the week can be assigned to any time slot, which you have set up. The vibration strength can be determined (via settings  → „Alarm Setting“), there are 5 selectable levels. Also, the flash (5, „LED Alert Light“) can be chosen for additional alerting.

Once set, the established alarm times will be kept and run within the »shake:Up« pad, independently from any connection to your smartphone.

To cancel an active alarm, briefly press the Snooze button. The alarm will repeat after the preferred time (this can be set under settings  → „Time Setting“). Pressing the Snooze button for 3 seconds will end all previous alarms until they repeat as intended.

You can also deactivate the alarm function entirely by sliding the on/off alarm switch (1) to the left (red switch side is visible). The status LED blinks 3 times in white when shutting down.

Notifications

Incoming calls and other notifications can be signalled by the pad. Within the app, under settings  , it can be determined which type of event is to be signalled by the pad, and at which vibration strength („Alarm Setting“). The flash (5) can also be chosen for additional alerting.

Please note: In order for this function to work, it is necessary that the Bluetooth connection between the phone and the pad is maintained.

An active Bluetooth connection is indicated by the status LED lighting up in blue every 25 seconds.

Status LED

The status LED informs you of various states of the pad:

When USB-cable is connected:

Constant red light:	Device is charging.
Constant green light:	Device is completely charged.

Other status information:

Flashes green 3 times when switched on.

Flashes white 3 times when switched off.

Flashing rapidly in blue:	Device is in Bluetooth pairing mode.
Blinking blue every 25 sec.:	Device has Bluetooth connection.
Flashes white 1 time:	Programming confirmation.
Flashes red every 5 seconds:	Low battery warning.
Flashes green 1 time:	Time synchronisation with phone.

Specifications

Diameter:	94 mm
Height:	27 mm
Weight:	132 g
Battery type:	Lithium-ion, 3.7 V, 1500 mAh
Charging time 0% to 100%:	2-3 hours
Use time per charge:	Approx 1 week, depending on usage intensity
Bluetooth version:	5.2
Selectable vibration levels:	5
Protection class:	IP 20

Specifications are subject to change without prior notice.

Bluetooth® is a registered trademark owned by Bluetooth SIG, Inc and is used under license.

Android is a brand of Google LLC.

Reset

If your »shake:Up« is not working properly or if you are still experiencing problems with the Bluetooth connection, you can try two different reset options:

Restart: Take a pointed object, insert it into the reset opening (7) and press the internal switch in the device. Your shake:Up will restart.

Reset to factory settings: Switch off the »shake:Up«. Switch it back on while holding down the snooze button – keep the snooze button pressed (approx. 10 seconds) until the status LED flashes red three times. The device will now be restored to its factory settings. Please note that this will also delete all alarm times, connection data and other preferences.

Information on product safety

Please observe these instructions to avoid damage to the unit and possible invalidation of the warranty.

The device is designed for use in living areas. The unit must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical shocks.

Do not drop the unit. Dropping the unit on a hard surface can damage or destroy it. Do not disassemble the device. Opening and modifying the unit will immediately void the warranty.

Strong interference signals from third-party devices can impair or completely prevent the function.

Environmental conditions

This unit is not suitable for outdoor use and should only be operated in a dry environment. It is not protected against splash water. Humidity and moisture can lead to failure of the unit.

Maintenance and cleaning

The unit is maintenance-free. If it becomes dirty, it can be cleaned with a soft, damp cloth. If necessary, use only commercially available alcohol-based disinfectants. Never use cleaning agents or disinfectants that contain acetone or hydrogen peroxide or that are bleaching, corrosive or abrasive. No liquid or moisture may penetrate the openings of the unit.

Service, warranty and warranty conditions

The unit has a high level of operational safety. If faults should occur despite proper operation, please contact your specialist dealer or contact Human-technik GmbH directly.

The warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so. Likewise, damage due to natural forces, such as fire and flooding, is not covered by the warranty.

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.
Always specify the serial number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system):

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow people by the correct disposal of this product.

Environment and health are endangered by a faulty disposal. Recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of this product from your local community, your local disposal company or your local dealer.

This device contains a battery and must not be disposed of in regular domestic waste. After use, the device can be returned free of charge to the seller or to collection points intended for this purpose (e.g. municipal collection points or in trade). This device is produced in compliance with current environmental regulations, complies with the statutory RoHS requirements and is free of any illegal substances.

Disposing of this device or its battery in fires, ovens or by mechanical crushing and cutting may cause fires or explosions. Exposing batteries to extremely high temperatures or low air pressure may cause an explosion or leakage of poisonous and/or flammable liquid or gas.

Compliance with the directives listed above is confirmed by the CE seal on the device.

RoHS: Directive 2011 / 65 / EC

WEEE: Directive 2012 / 19 / EC

RED: Directive 2014 / 53 / EC

CE compliance declarations are available on the Internet at [www.humantechnik.com /service](http://www.humantechnik.com/service).

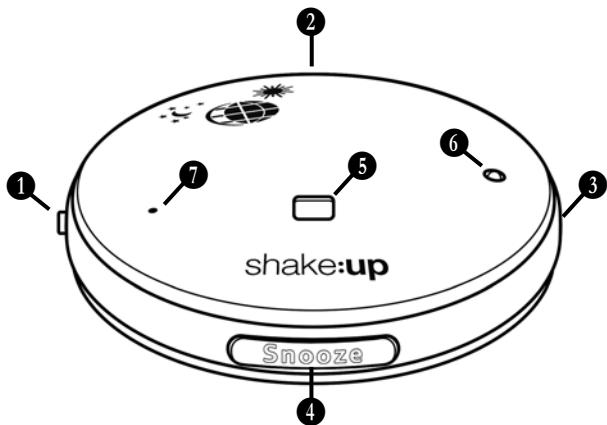
Humantechnik's UK authorised representative:

Sarabec Ltd.

15 High Force Road

MIDDLESBROUGH TS2 1RH, United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments. UK declaration of conformity available from: Sarabec Ltd.

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Humantechnik GHL AG