HUMANTECHNIK

earis XS

Pocket Receiver Set

DE

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Seite 2

EN

User manual

Page 24

FR

Mode d'emploi

Page 46

NL

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Pagina 68

ES

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Página 90

IT

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Pagina 112

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Index	Page
Important safety notices	25
Safety notes on the lithium-polymer rechargeable battery	26
Introduction	26
Contents of package	27
Product overview	28
Connecting the transmitter to an audio source.....	31
Connecting the transmitter to digital-out (S/PDIF / Toslink).....	31
Connecting the transmitter to RCA sockets	32
Connecting the transmitter to a headphone jack socket	32
Connecting the transmitter to a SCART socket.....	33
Powering the transmitter.....	33
Using the rechargeable battery	34
Using “earis XS”	35
Using the “earis XS” pocket receiver	36
Audio source selection	38
Adjusting the volume	38
Adjusting the balance	38
Set listening profile for sound improvement.....	39
Mute-Function	39
Pairing transmitter and receiver.....	40
Important notes	41
Warranty.....	41
Environmental information / disposal.....	41
Troubleshooting	43
Specifications	45

Important safety notices

Before setting up and using the device, please read these instructions carefully and thoroughly.

- Keep this user manual in a safe place so that it can be accessed by other users at all times.
- Do not use the device if it is obviously damaged.
- Only use the device in locations where 2.4 GHz radio technology is permitted.
- The device and in particular the power supply unit are only intended for operation in dry indoor areas. Operation is only guaranteed within room temperatures of 5°C to 35 °C.
- Keep the product, packaging and accessories away from children and pets to avoid accidents.
- Always handle the device and accessories carefully and do not place them in dirty or dusty areas.
- Keep the transmitter and receiver clean.
- Paints and furniture polishes can chemically attack the transmitter's rubber feet and cause surface damage to your furniture.
- Use only the provided power supply unit to power the transmitter.
- In the event of several days' absence, we recommend unplugging the power supply unit from the mains electricity socket.

Attention:

Your receiver can be set to a very high volume. To protect your own hearing or the hearing of people with regular hearing capabilities, especially children, from damage caused by excessive volume, please observe the following instructions:

1. Set the receiver to a low volume level before using it.
2. During use, adjust the volume to a level that is comfortable for you.
3. If you have a pacemaker, please check with your physician or the manufacturer of the pacemaker before using this product.
4. If you use other medical devices, use the device only in accordance with the safety instructions of your physician or the manufacturer.

Safety notes on the lithium-ion rechargeable battery

The battery used in the “earis XS” is a lithium-ion battery manufactured in accordance with the latest standards and safety regulations. Misuse or improper treatment can damage the battery, in which case the Humantechnik Group assumes no liability.

In extreme cases there is a risk of:

- development of heat
- development of fire
- explosion
- smoke or gas emission

- Do not expose the battery to temperatures above 70°C, e.g. by sunlight or throwing it into fire.
- Do not expose the battery to water and condensation.
- Do not short-circuit the battery.
- Do not disassemble or deform the battery.
- Store the device / battery in a cool and dry place at room temperature.
- Only use the chargers provided for the battery.
- When not in use for a prolonged time, recharge the battery in the unit regularly (every 6 months for approx. 1 hour).
- Charge the battery only at an ambient temperature of 10 °C to 40 °C.
- If the device is obviously defective, do not recharge the battery.
- Refer to the information regarding warranty and disposal.

Introduction

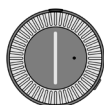
This digital wireless pocket receiver uses the latest digital radio technology and enables sound amplification. Enjoy crystal clear stereo and TV sound anywhere in the house.

This digital wireless pocket receiver is intended to be used as assistive listening that can be used help the hearing impaired.

Please read this user manual carefully in order to set up the device correctly and to become familiar with all the possibilities it offers.

Contents of package

1 "earis XS" pocket receiver

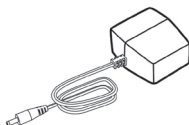
1 "earis XS" transmitter


1 earphones
(Europe only)


1 rechargeable battery

1 power supply

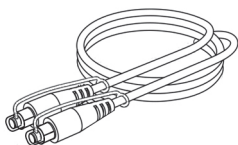1 Toslink cable

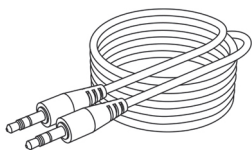1 audio connection lead with 3.5 mm stereo
jack plugs

1 quick guide

1 user manual

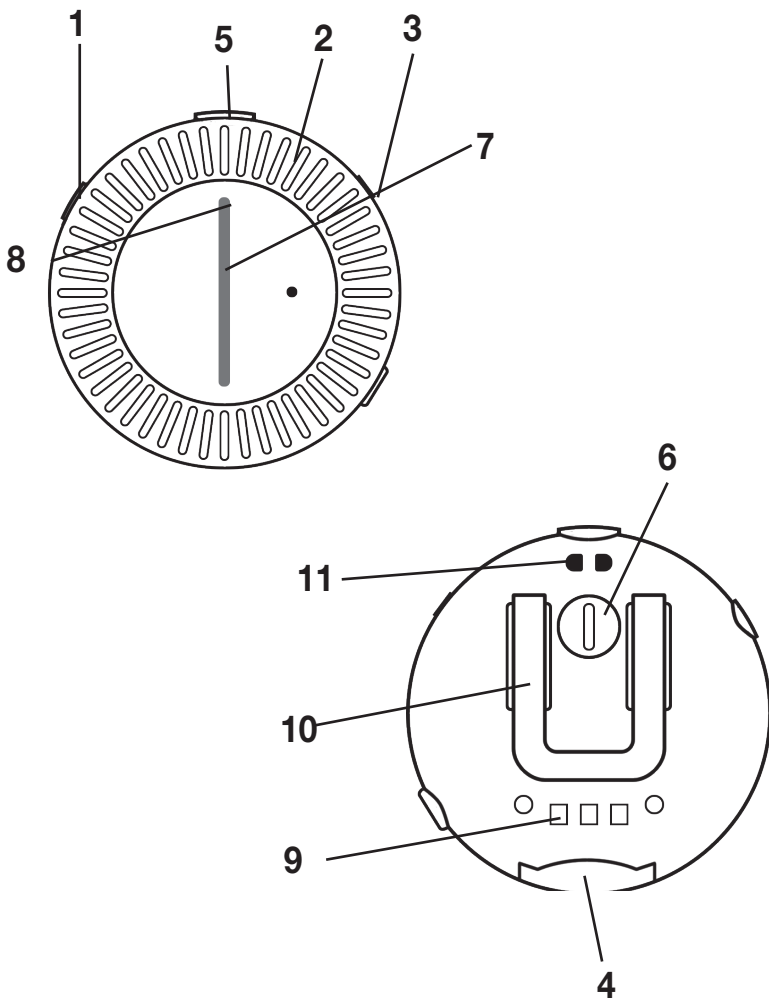
1 warranty card


Product overview

Overview: Pocket receiver (PR)


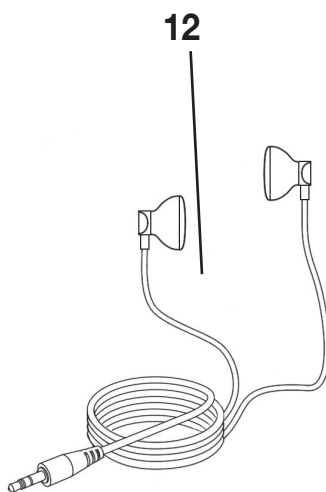
1. On / Off button
2. Volume control
3. 3.5mm stereo jack socket
4. Battery compartment
5. Status-LED / function button
6. Balance adjuster
7. Volume indicator
8. Battery status
9. Charging contacts
10. Belt clip
11. Eyelet for carry thread
12. Earphones
(Europe only)

Overview: Transmitter

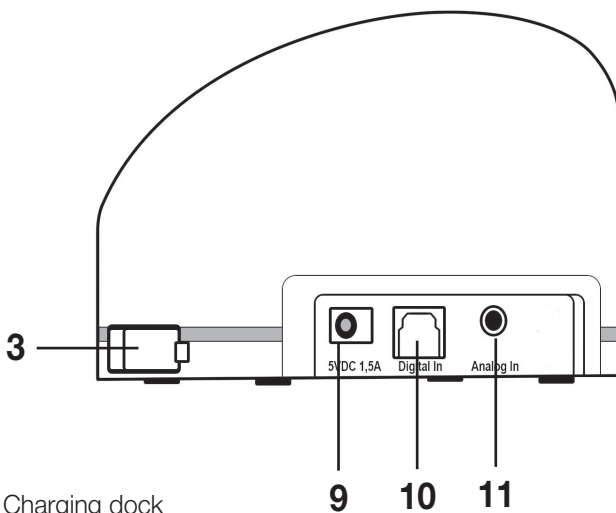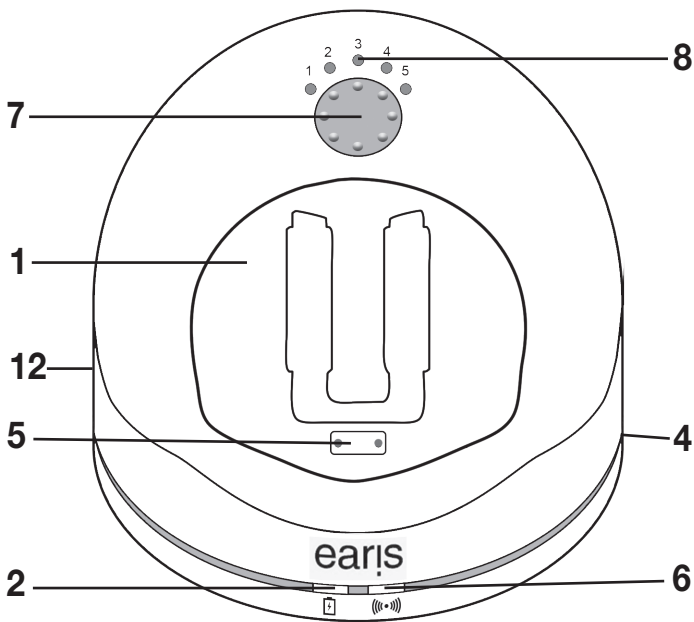1. Charging dock
2. Status LED for receiver in charging dock
3. Charging compartment for spare rechargeable battery
4. Status LED for spare rechargeable battery
5. Charging contacts with support magnet
6. Status LED
7. Tone adjustment / pairing button
8. Tone setting indicator
9. Connection socket for power supply
10. Optical-digital Toslink connector
11. Jack socket for connecting analogue audio sources (blue)
12. Analogue / digital selector switch

Overview: Status LED on transmitter

Status LED

dark

white light

white light flashes fast

flashes 4x in white

white light flashes 4x slow

Transmitter.....

... is switched off

... is switched on and transmitting

... is in pairing mode

....has no PCM signal at digital input

Connection to receiver lost

Overview: Status LED on receiver

Status LED

dark

constant green light

volume indicator bar flashes

rapidly

red LED on top

Receiver.....

... is switched off

... is switched on and receiving

... is in pairing mode

....Battery is empty soon

Overview of audible alerts

Audible alert

during use: 1 beep

every 30 seconds

when altering the volume:

1 beep

when altering the volume:

2 beeps

during use: 5 beep

every 20 seconds

Signification

battery is almost depleted

maximum volume reached

minimum volume reached

receiver is out of transmission range
of the transmitter

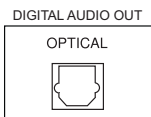
Connecting the transmitter to an audio source

Turn off the audio source before connecting the transmitter. Place the transmitter near the audio source. Do not place the transmitter in the immediate vicinity of thick reinforced concrete walls and other radio-transmitting devices, as the transmission range could be reduced.

You have several possibilities to connect “earis XS” to a sound source (e.g. TV, stereo set etc.).

Check the options of connection of your audio source.

Connecting options:


- S/PDIF digital output (Toslink)

- RCA sockets

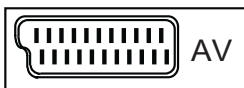- Headphone jack

- SCART socket

Connecting the transmitter to digital-out (S/PDIF / Toslink)

For best audio quality, connect the transmitter to your audio source using the Toslink optical (digital) cable.

Please note that the digital input can only transmit audio sources in PCM / Stereo standard.

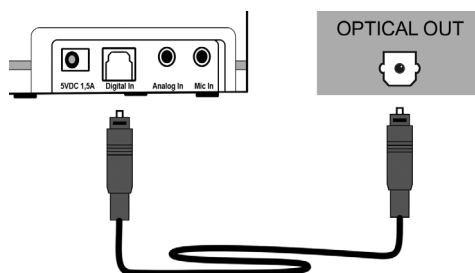
Refer to the manual of the connected audio source to learn how to set up the digital output.

Audio playback is not possible if a combined digital signal (bitstream) is used. The status LED on the transmitter will flash twice, with a pause in between, if there is no usable digital signal input.

Remove the protective caps on the two Toslink connectors on the cable. Plug the Toslink optical cable into the transmitter's Digital In connector and connect it to your audio source. The analog/digital switch on the transmitter side must be set to "Digital".


Connecting the transmitter to RCA sockets

Good audio quality can be achieved by connecting the transmitter to your audio source using the analogue audio connection lead. The Cinch adapter cable is not included in the scope of delivery. It can be purchased from your dealer.

Plug the supplied audio connection cable into the transmitter's blue "Analog In" socket. Connect the RCA adapter cable audio connection lead. Plug the RCA connectors into the RCA sockets of your audio source. Make sure that the white connector is connected to the white socket of the audio source and the red connector is connected to the red socket of the audio source.

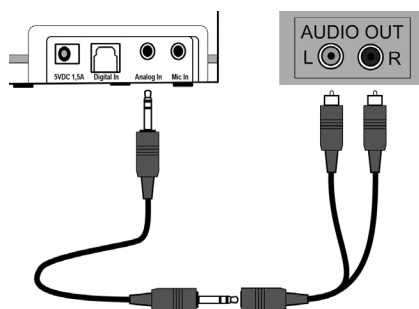The analogue/digital switch on the transmitter's left side must be set to "Analog"


Connecting the transmitter to a headphone jack socket

Plug the supplied audio connection lead into the transmitter's blue "Analog In" socket. Plug the audio connection lead into the headphone jack of your audio source.

Set the volume of your sound source to a medium level, where possible.

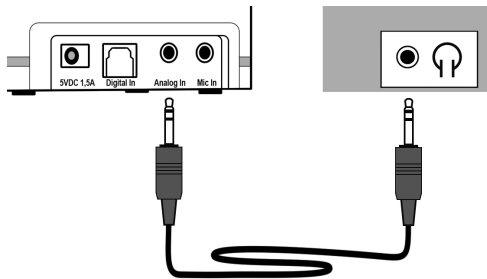There are TV sets that mute the speakers when devices are connected to their headphone jack. On some television sets, this automatic muting can be turned off via the TV settings menu. Alternatively, use another option for connection. The analog/digital switch on the transmitter's left side must be set to "Analog".

Connecting the transmitter to a SCART socket

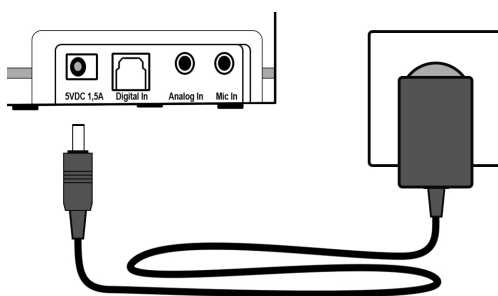
The SCART adapter is not included in the scope of delivery and can be purchased from a specialist dealer.

Plug the supplied audio connection lead into the transmitter's blue "Analog In" socket. Connect the SCART adapter with the audio connection lead.

Plug the SCART adapter into the SCART socket of your audio source. The analog/digital switch on the transmitter's left side must be set to "Analog".

Powering the transmitter


Connect the power supply unit to the transmitter with the yellow "5V DC 1.5A" power socket. Then plug the power supply unit into an electrical mains outlet.

Using the rechargeable battery

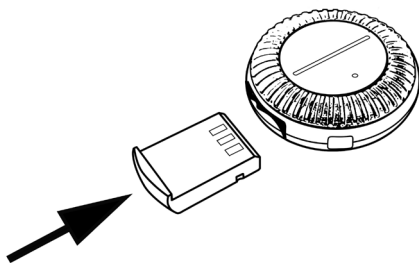Insert the supplied battery into the receiver, until it clicks into place (the battery only fits one way)


Charge the receiver completely and without interruption before first use. A complete charge will take about 2 hours.

The battery status on the receiver lights up red when the battery level is low. When system is in use and the battery in the receiver is low, the status LED on the receiver flashes RED constantly, and you will hear a beep every 30 seconds. After a few minutes, the receiver will automatically switch off.

To charge the battery, place the receiver in the charging dock of the Transmitter. The LED bar must point upwards, magnets keep the receiver in the correct charging position.

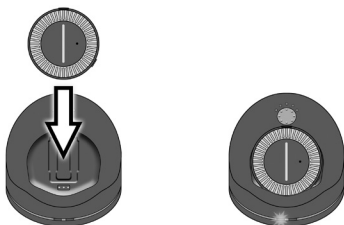The charging status LED on the front of the receiver will illuminate red, the charging status LED will switch to green when the battery is fully charged.

We recommend that you always place the receiver in the charging dock of the transmitter when the system is not in use, this will always keep the battery in optimum condition. The intelligent charging circuit within the system will prevent overcharging of the battery.

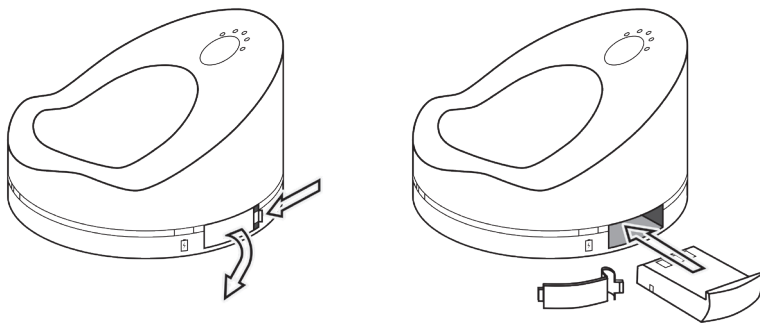
A second battery can be charged in the transmitter, additional batteries are available from your dealer.

The replacement battery can be inserted into the charging slot on the right side of the transmitter, please remove the cover over the charging slot.

The battery only fits one-way, extreme force is not required to insert it into the transmitter.

During charging, the charging status LED for the spare battery lights up red when the battery is fully charged this indicator changes to green.

If you have a second battery we recommend that you swap the batteries once a week, this exercises the batteries and allows them to reach their maximum lifespan.

Using “earis XS”

Attention:

Your receiver can be set to a very high volume. To protect your own hearing or the hearing of people with regular hearing capabilities, especially children, from damage caused by excessive volume, please observe the following instructions:

1. Set the receiver to a low volume level before using it.
2. During use, adjust the volume to a level that is comfortable for you.
3. Do not expose yourself to high volume for an extended amount of time.

Using the “earis XS” pocket receiver

With the “earis XS” pocket version it is possible to connect the included headphones. Further accessories can be connected:

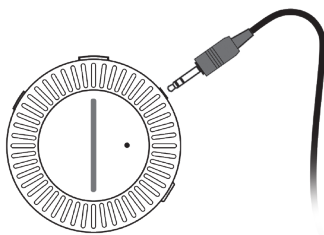
- Induction loop
- Underchin headphones
- Induction hooks monaural
- Induction hooks binaural
- DAI-Audio cable monaural
- DAI-Audio cable binaural

Connecting a neckloop to the “earis XS” pocket receiver

You can only use the induction loop if you have a hearing aid with an induction coil (T-coil). The induction loop is not included. It can be purchased from your dealer.

Connect the 3.5 mm jack plug of the induction loop with the jack socket of the pocket receiver.

Set your hearing aid to the T-position or loop program to hear the sound. If your hearing aid does not have a T-position or if you wish to use a cochlear hearing aid, please contact your hearing care professional.

Connecting headphones to the “earis XS” pocket receiver

Connect the 3.5 mm jack on the headphones with the jack socket on the pocket receiver.

To use the “earis XS” pocket receiver, proceed as follows:

1. Switch on your sound source (e.g. TV). If “earis XS” is connected via the headphone output, you should set the volume of the sound source to a medium volume. This is not necessary if Earis XS is connected via Toslink or Cinch.
2. The receiver switches itself on automatically when you take it out of the charging dock. When the receiver is on, the green status LED on top of the receiver is lit.
Use the volume control on the receiver to adjust the volume to a low level to protect your hearing. A low level of volume is indicated when only a few LEDs on the volume indicator are lit. The more LEDs on the volume indicator light up, the higher the set volume.

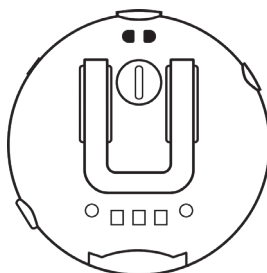
3. The white status LED on the transmitter lights up as soon as the transmitter is in contact with the receiver and a transmission is established.
4. Adjust to the desired volume.
5. The pocket receiver switches off automatically when you place it in the charging dock. We recommend to always return the receiver to the transmitter's charging dock so that the battery is always charged. If no audio signal reaches the transmitter for more than 15 minutes, it will switch off automatically.
6. As an alternative, the pocket receiver can also be switched on and off by prolonged pressing of the ON/OFF switch on the upper left side of the receiver. If no audio signal reaches the pocket receiver for more than 15 minutes, it will switch off automatically.

For best performance and range, place the receiver correctly. If possible, the receiver should have a line of sight with the transmitter.

If possible and for ease of use, hold the receiver in your hand while having visual contact to the transmitter.

The clip on the rear side of the receiver can be used to fasten the “earis XS” pocket receiver to your belt or clothing.


You can also wear the receiver on the optional neckloop by attaching the receiver to the loop of string coming out of the bottom of the neckloop's housing. Thread this loop of string through the eyelet in the receiver and form a sling by putting the entire neckloop through the string's loop.


Audio source selection

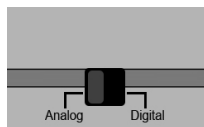You have the possibility to connecting a maximum of 2 audio sources to the transmitter, for example an analogue stereo system and a digital TV set. Use the “Analog/Digital” input selection switch on the left side of the receiver to select the desired audio source.

Adjusting the volume

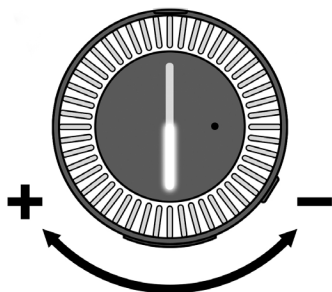
Use the silver volume control on the front of the receiver to easily adjust the desired volume by turning it. The LED bar indicates the set volume. A low level of volume is indicated when only a few LEDs on the volume indicator bar are lit. The more LEDs on the volume indicator light up, the higher the set volume.

Adjusting the balance

The balance control allows the volume distribution between the right and left ear. Adjust the balance so that you hear equally well in both ears.

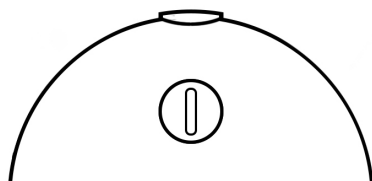Please note that, when using a neckloop, the balance control has no effect as the neckloop can only reproduce sound in mono.




Set listening profile for sound improvement

With our hearing profiles you have the possibility to find an optimal setting for your hearing sensation. You can choose the most comfortable profile for you from 5 sound profiles. The set profile is saved.

The following profiles can be selected:

1. strong sound alteration in the low frequency range
2. minor sound alteration in the low frequency range
3. no change
4. minor sound alteration in the high frequency range
5. strong sound alteration in the high frequency range

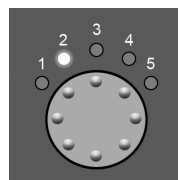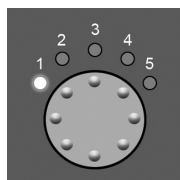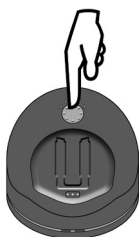1. Press the tone control button on the top of the transmitter to select the desired profile. You can identify the set profile on the transmitter by the 5 sound profile indicator LEDs.


You can connect a maximum of two receivers to one “earis XS” transmitter. The transmitter calculates the sound profile, while the adjusted sound profiles are stored in the receiver.

The transmitter always calculates the set sound profile of the receiver that is switched on first.

Mute-Function

The receiver has a Mute function which will allow the user to interrupt the sound coming from the system without removing the receiver.

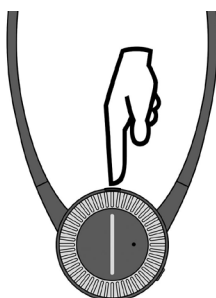1. To use this feature, a short press of the function button on the top of the receiver will cause the LED indicator to flash briefly, sound is then interrupted.
2. To switch the sound back on simply press the function button again, the LED indicator will light up constantly and the normal sound will resume.

Pairing transmitter and receiver

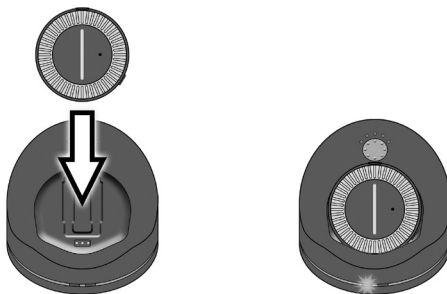
For “earis XS” to function properly, transmitter and receiver must be paired. As a rule, this has already been done during production and is carried out as a standard on delivery.

However, if you want to use another receiver or perhaps have to replace one later, it must first be paired with the transmitter.

A maximum of 2 receivers can be used simultaneously with one transmitter.

There are 2 ways to pair a receiver with the transmitter:

1. When inserting a receiver into the charging dock of the transmitter for at least 15 seconds, the according receiver and the transmitter will be paired automatically.


Or 2. Press and hold the tone control button until the status LED on the front of the transmitter starts flashing rapidly.

Underchin receiver: Wear the underchin receiver to switch it on.

Pocket receiver: You must remove the pocket receiver from the charging dock to turn it on.

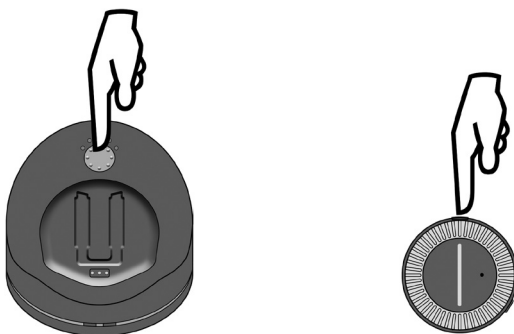Press the function button on the top of the receiver until the flashing status LED on the transmitter lights up constantly. This indicates that transmitter and receiver are now paired.


Important notes

The equipment is designed for listening within domestic environments. Other uses are outside the scope of use for this product. Damage or contamination caused to the equipment, caused by use not in accordance with these instructions, is excluded from the warranty.

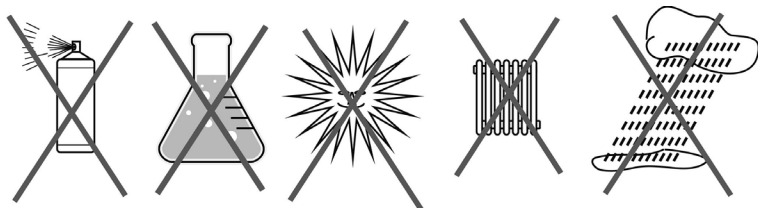The devices are not waterproof and should be kept away from all liquids. Do not place them near any heat sources. Do not place any items on top of the devices which may cause them to malfunction.

Warranty

"earis XS" is a highly reliable product. Should a fault occur, contrary to proper use, please refer to the troubleshooting guide. If this does not resolve the problem, please contact your retailer or the manufacturer directly for further help before returning any items.

The warranty covers free repair as well as free return shipment, where applicable.

We recommend to return the items in the original packaging, so please keep it for the length of the warranty period.

This warranty does not cover misuse, improper handling or attempted repair by unauthorised persons (broken device seal). Warranty repairs will only be carried out if proof of purchase is provided to the retailer or manufacturer.

Always specify the product number in any event.

Environmental information / disposal

Disposal of used electrical and electronic devices (to be applied in countries of the European Union and other European countries with a separate collection system for these devices).

The symbol on the product or on its packing indicates that this product must not be treated as normal household waste, but that it must be returned to a recycling point of acceptance for electrical and electronic devices or that it must be disposed in a Yellow Bin. You protect the environment and the health of your fellow man by contributing to the correct disposal. Environment and health are endangered by incorrect disposal.

Material recycling helps to reduce the consumption of raw materials.

Additional information about recycling of this product can be obtained from your community, the municipal disposal companies or the shop where you bought the product.

Battery

The provided battery can be recycled. Please dispose of the battery through battery containers or a specialist shop. Please dispose only discharged batteries to guarantee environmental protection.

These devices comply to the requirements of the following EC directives:

- 2011/ 65/EU + 2015/863/EU RoHS-directive
- 2012/19/EU WEEE-directive
- 2014/53/EU RED-directive

Conformity with the above-mentioned directives is confirmed by the CE mark on the device. CE declarations of conformity are available on the Internet at www.humantechnik.com/service

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments.

The full declarations of conformity for this device are available from Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH. UK.

This devices adheres to the requirements of the following IC directives:

RSS-247 Issue 2 (2017-2)

Technical changes reserved.

Troubleshooting

Symptoms	Cause	Solution
Charging status LEDs do not light up, despite batteries inserted in the transmitter charging stations.	The transmitter is without power from the power supply unit.	Check that the power supply unit is properly connected and that the mains electricity outlet is working properly.
Charging status LED does not light up, despite a battery being inserted.	No contact between battery and charging station in the transmitter.	Insert the batteries correctly and completely into the transmitter. Check if the contact areas are clean.
Charging status LED for the receiver does not light up, despite the receiver being put in the charging dock.	No contact between the receiver battery and the charging dock of the transmitter.	Make sure that the receiver is correctly placed in the charging dock.
The receiver does not work, no sound transmission.	No battery placed in the receiver.	Insert the battery in the receiver.
	The transmitter is not properly connected to the TV- or stereo set.	Check the cables and connectors between the transmitter and the audio source.
	The TV- or stereo set is not switched on or the output has not been configured properly.	Switch the audio source on or check the settings.
	The pairing has been lost	Pair the transmitter and the receiver again.
The receiver does not work, no sound transmission.	The receiver is not switched on. (Pocket Version)	Switch the receiver on, using the ON/OFF switch on the side of the device. (Pocket Version)

Symptoms	Cause	Solution
	Mute function activated	Press the function knob to deactivate the mute function
	The receiver battery is depleted or not charged.	Charge the battery or insert a charged battery.
Sound transmission is poor, noisy or disturbed.	The receiver is too far from the transmitter.	Bring the receiver closer to the transmitter.
Sound transmission seems distorted, muted or disturbed.	The battery is almost depleted.	Charge the battery or insert a charged battery.
The sound transmission is briefly interrupted and then works again	The receiver is too far from the transmitter. The surroundings hinder transmission (walls, objects).	Bring the receiver closer to the transmitter.
The battery lasts only a short while.	Battery is not fully charged or defective and therefore no longer able to work.	Charge the battery, insert a fully charged battery or a new fully charged battery.

Specifications

Earis XS Set

Modulation:	Pulse-code modulation PCM Analog-over-digital
Carrier frequency:	2.4GHz~2.5GHz
Signal-to-noise ratio:	-87 dB
RF output power:	max. 10dBm Class 1
Audio latency:	< 47ms
Working range:	ca. 70m
Temperature range:	0°C bis 45°C
Power:	5 Volt DC / 1.5A
Power supply:	100-240 Volt 50-60 Hz 300mA

Earis XS Pocket Empfänger

Connector:	stereo jack socket min. 16Ohm
Power:	Akku AP18A 3.7V 400mAh Lithium-Ion
Working time:	ca. 9h with headphones ca. 7h with neck loop
Charging:	ca. 2h
Weight (incl. rechargeable battery):	41g
Dimensions:	63mm*24mm
Temperatur range:	0° C – 45° C

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