**DE**

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Seite 2

Komfort-Kopfhörer

**EN**

**User manual**

Page 4

Comfort headphones

**FR**

**Mode d'emploi**

Page 10

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**NL**

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Pagina 14

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**IT**

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Pagina 18

Cuffie comfort


**ES**

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Página 22

Auriculares de confort
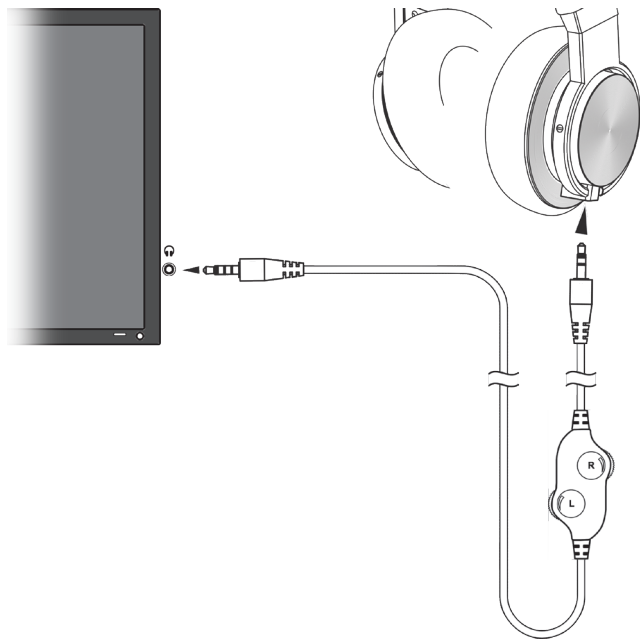
## Anschluss

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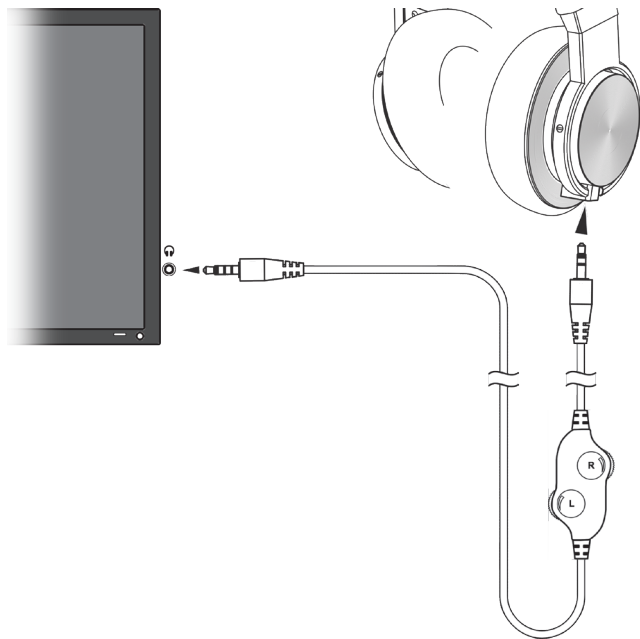
## Connection

Congratulations on purchasing the comfort headphone »LH-060TV«. With its Neodymium magnet technology, the LH-060TV delivers a pleasant and powerful sound. The soft earpads enable a comfortable use over hours.

Please read this manual in order to correctly connect and use the item.

### Scope of delivery

Please check if the following parts are included in the package:

- Headphones »LH-060TV«
- Connection cable with volume adjuster
- Jack adapter 6,3 mm
- User manual
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

### Connection

Connect the jack plug which is closest to the cable's volume adjuster with the headphone's connection socket.

Connect the second jack plug with the connection socket of your TV set or any other audio source (mobile phone, MP3-Player, stereo set), which features a headphone jack port.

Use the supplied adapter if needed.

Set the volume on the TV-set or other audio source to the highest comfortable level. A separate adjustment for the left or right side can be made over the volume adjuster on the connection cable.

### Technical data

Output performance:	20 mW
Diameter of magnetic coils:	50 mm
Sensitivity:	113 dB
Impedance:	24 $\Omega$
Frequency range:	20Hz ~ 20KHz
Length of cable:	5,5 m

### Using the headphone

This device satisfies the following EC-directives:

- 2014/35/EC Low voltage directive
- 2011/65/EC RoHS directive
- 2012/19/EC WEEE directive
- 2014/30/EC EMC directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humantechnik.com/service](http://www.humantechnik.com/service).

Technical specifications subject to change without prior notice.

## Maintenance and care

The »LH-060TV« does not require any maintenance. If the headphone does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not expose it to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

### Note:

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Warranty

The »LH-060TV« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so please keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on or within the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

## Disposal

of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

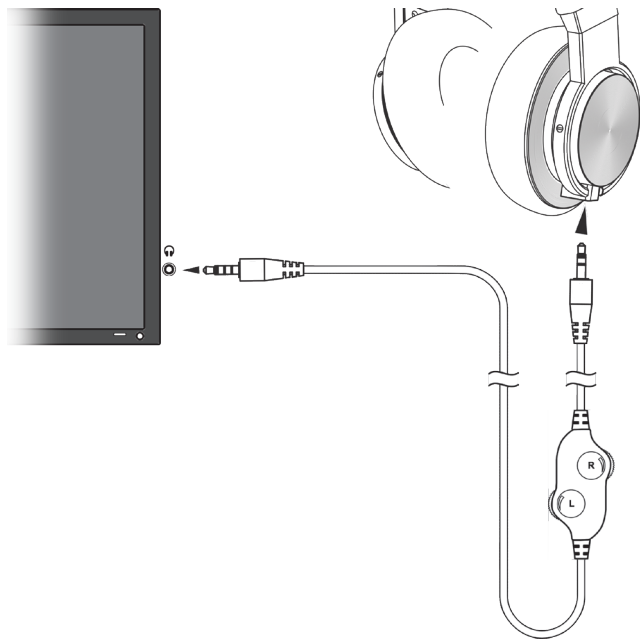
The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units.

You protect the environment and health of your fellow man by the correct disposal of this product.

Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

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**Sarabec Ltd**